



**BEFORE USING THE PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY. IMPORTANT!**

This booklet is designed to assist in using this product. It is not a reference to medical techniques. This unit was designed, tested and manufactured for single patient use. Reuse or reprocessing of this unit may lead to its failure and subsequent patient injury. Reprocessing of this unit may create the risk of contamination and patient infection. Do not reuse or reprocess this device.

**Description**

The ClearPath Irrigator is a single use disposable adaptor, attachable to endoscope working channels. This device is designed for use with Pentax endoscopes (purple adaptor) and with Olympus, Fujinon endoscopes (green adaptor).

**Indications**

The ClearPath Irrigator is an irrigation / biopsy adapter set used to cover the opening to the biopsy / suction / irrigation channel of Pentax, Olympus and Fujinon endoscopes. It provides access for endoscopic device passage and exchange, helps maintain insufflation, minimizes leakage of biomaterial from the biopsy port throughout the endoscopic procedure, and provides access for irrigation.

It is for use only by trained medical personnel located in hospitals, clinics, and doctors' offices.

**Warnings and precautions**

1. This equipment should only be used under the supervision of a trained physician in a medical facility. Do not use in any other location or for any purpose other than the intended application.
2. Verify mechanical and electrical compatibility of devices from different manufacturers prior to using them together in a procedure.
3. This device is provided clean and is intended for use in a SINGLE procedure only. DISCARD AFTER USE.

**Instructions**



Attaching the ClearPath Irrigator to the endoscope (refer to figure 2):

1. Replace the biopsy valve on the working channel with the ClearPath Irrigator adaptor (refer to figure 1).
2. After affixing the Irrigator on the biopsy port rotate to each side, make sure the adaptor sits well.

*Connecting the ClearPath Tubing to the ClearPath Irrigator:*

3. Remove the cap from the distal end of the check valve attached to the adaptor.
4. Connect the Luer lock on the tubing side to that on Irrigator side (refer to figure 2).
5. Turn the ClearPath Controller ON.
6. Before starting the procedure, operate the irrigation pedal and verify, visually, that the system functions properly.

**Notice:** Priming required - when operating the irrigation initially, water will start flowing after 3-4 activations of the foot pedal.

*During the procedure:*

7. Operate the irrigation by pressing on the **BLUE** "Irrigation" pedal.

*After the procedure:*

8. Disconnect the ClearPath Irrigator from the tubing
9. Remove the ClearPath Irrigator adaptor from the endoscope and discard it.

CE  
0470

**Store at room temperature.**

**Avoid prolonged exposure to elevated temperatures.**

EG-C-RD 0315-03



MedNet GmbH, Eschstrasse 19, D-48167  
Munster, Germany  
Tel: +49 2506 8644-0, Fax: +49 2506 8644-22  
Email: [info@medneteuropa.com](mailto:info@medneteuropa.com)



EasyGlide Ltd.  
30 Haella St. Kfar Truman, Israel 73150  
[www.easy-glide.com](http://www.easy-glide.com)

**Caution:** Federal (USA) law restricts this device to sale by or on order of a physician.